Stollingsapparaat

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Testing Citrated Whole Blood Samples

Citrate Activated Partial Thromboplastin Time (aPTT) and Citrate Prothrombin Time (PT)

Collect blood samples **at least 5 minutes prior** to testing to allow for adequate equilibration of the sodium citrate with the sample.

Test all citrated samples **within 2 hours of collection**. Keep the sample at room temperature.

1. Remove IDEXX Coag Dx™ Citrate PT and/or Citrate aPTT cartridges from refrigerated storage and allow them to come to room temperature (15°–30°C/59°–86°F). This takes 30–60 minutes. (**Do not** open the pouch prior to use.)
2. Prepare the venipuncture site by shaving fur (if necessary) and cleansing with alcohol. Allow the site to air-dry completely, as alcohol interferes with the coagulation assay.
3. Use **one** of the following methods to collect the blood sample:
 - Collect the sample directly into a 3.2% sodium citrate evacuated tube.
 - Collect the sample in a plastic (not glass) syringe and transfer it to a 3.2% sodium citrate evacuated tube.

Note: Fill the citrate tube to its stated draw volume.

4. Mix gently by inverting the tube at least 4 times.
5. The message "Insert Cartridge" displays on the Coag Dx Analyzer. Open and insert a room temperature IDEXX Coag Dx™ Citrate PT or Citrate aPTT cartridge into the analyzer to warm (Figure 1).
6. Press the **ID** key and enter the client ID, patient ID and species.

7. Gently invert the evacuated tube again, 4 times, and then collect the sample from the tube using the disposable plastic pipette (Figure 2).
8. Dispense one drop of blood into the sample well of the cartridge (Figure 3).

Figure 2: Collect sample from 3.2% sodium citrate evacuated tube

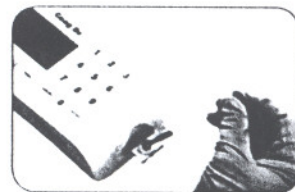
Figure 3: Add sample to center of well

Note: A sufficient quantity of blood must be added to the center of the sample well to fill it slightly over the top (Figure 4, sufficient vs. Figure 5, insufficient). If a large drop of blood extends above the center sample well (Figure 6), use a pipette to push the extra blood into the outer sample well (Figure 7).

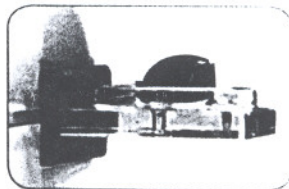
Figure 4: Sample is sufficient; meniscus protrudes slightly above sample well

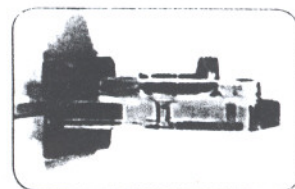
Figure 5: Insufficient sample

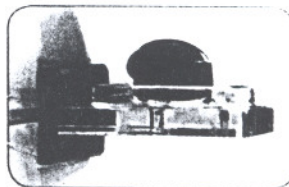
Figure 6: Sample too large

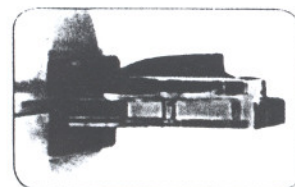
Figure 7: Sample pushed into outer well

9. Press the **Start** key. When a clot is detected, the analyzer beeps once. Final results of the test are displayed on the screen.
10. Remove the cartridge. Discard used needles, syringes, pipettes and cartridges in the appropriate waste containers.

IDEXX
LABORATORIES

One IDEXX Drive
Westbrook, Maine 04092 USA
idexx.com

06-14004-00

© 2007 IDEXX Laboratories, Inc. All rights reserved.
IDEXX VetLab and Coag Dx are trademarks or registered trademarks of
IDEXX Laboratories, Inc. in the United States and/or other countries.

USA/Canada/Latin America 1-800-248-2483

Europe 00800 1234 3399 • Australia 1800 655 978

New Zealand 0800-102-084 • Asia 886-2-28883336,230

IDEXX Coag Dx[™] Analyzer

Reference Interval Guidelines

Elapsed time in whole blood seconds:

	Canine	Feline	Equine
Citrate PT	11-17	15-22	15-20
Citrate aPTT	72-102	65-119	Not validated
PT	11-14	13-22	15-18
aPTT	60-93	60-115	Not validated

• Reference intervals for citrate cartridges were validated using 3.2% sodium citrate evacuated tubes.
• Reference intervals are subject to change. Refer to the cartridge package insert for current reference intervals.

IDEXX Technical Support

U.S./Canada/Latin America 1-800-248-2483

Europe 00800 1234 3399

Australia 1-800-655-978

New Zealand 0800-102-084

Asia 886-2-2888-3336, 230